## ultraRESCUE BOARD

Patientenumlagerung / Patient transfer

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*Please read the instructions for use completely before use.  
Always store the instructions for use near the medical device.*

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### Conventional use

*The Roll board (**Fig. 1 folded**) is used when a patient needs to be transferred, e.g. from a stretcher to a bed. Due to the way the Roll board works, it is no longer necessary to lift or carry the patient.*

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### Description

*The board can be folded in the middle and is equipped with a cover made of blue nylon fabric. It has handles on the short sides that allow easy carrying when the board is folded. The cover is open on the sides and can be easily exchanged.*

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### **Technical data**

Measurements (L/W/H): Long/foldable 100 x 45 x 4 cm · Weight: approx. 1,6 kg · applicable to 110 kg patient-weight · Board: Polyester fabric, color: white, one side coated, plastic plate with a pad of foam inside · Cover: Nylon fabric, blue

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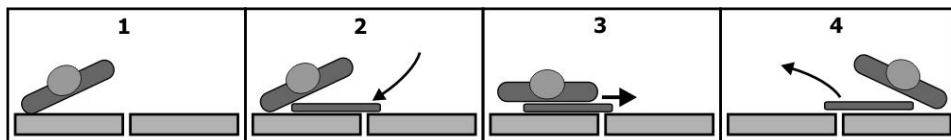
### **Preparation**

Between the two beds, there should be a height-difference of approx. 4-5 cm. The patient should be move from the higher on the lower bed. The transfer also is without any problems at the same height of the two beds. The transfer on an elevated surface also is possible, but it requires a higher effort.

Both beds must put together as densely as possible. The split between both beds should not exceed 10 cm. Both beds must be prevent against slipping or inadvertent rolling.

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### **Application**

1. Lift on the patient to one side.
2. Open the board (Art. 5230) and put it lengthwise under the patient.
3. Put the patient back again so that he lies on the board. It is important that buttock and shoulders are place on the cover. Now, the patient with the Sliding board can push or pull on the new bed. The board glides on the new bed as well.
4. Lift on the patient after the transfer to the other side and take the board away.  
For safety reasons the transfer should be conducted of two people. For big or heavy patients is necessary for a risk-free transfer in accordance with more people.

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### Liability for material defects:

*We achieve a cost-free rectification of defects or a cost-free substitute for material or manufacture-mistakes (material defects) from date of production within 12 months. Damages are not implied, that are originated through mechanical (e.g. Pricks) or chemical (e.g. sharp disinfectant) influences during the use or the storage.*

*We are not liable for direct or indirect damages of all type at people or objects, which are create through improper use, contempt of the directions for use, or due to the not possible use because of omitted or not finished repairs.*

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### Tasks and duties for the operation of a medical device

*The operator of a medical device is responsible for the commissioning and operation of the product. This means that the legal provisions must be observed and implemented. The operator must create*

*the organizational requirements for the intended use of the product and ensure that the operating personnel have the necessary expertise and reliability. The operator's duties also include ensuring the functionality and proper condition of the product.*

*Maintenance may only be entrusted to persons who have the necessary expertise. The exact wording of the requirements of all responsibilities of the operator of medical devices can be found in the current legal provisions of the country in which the product is used.*

*For disposal or recycling of the product, please contact your local disposal company. With free return to our company, the product will be environmentally friendly recycled.*

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Roll board foldable, cover blue

Art.-Nr. SAN-8020-S

Replacement cover for Roll board foldable, blue

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**B.u.W. Schmidt GmbH**  
Porschestraße 29  
D - 30827 Garbsen  
Telefon +49 (0) 5131 / 60 08  
Telefax +49 (0) 5131 / 9 63 93  
E-Mail: [mail@buwschmidt.com](mailto:mail@buwschmidt.com)  
Internet: [www.buwschmidt.com](http://www.buwschmidt.com)

**ultraMEDIC GmbH**  
Im Bruch 11-15  
D – 56567 Neuwied  
Telefon +49 (0) 2631 / 96983 0  
Telefax +49 (0) 2631 / 96983 33  
E-Mail: [info@ultramedic.de](mailto:info@ultramedic.de)  
Internet: [www.ultramedic.de](http://www.ultramedic.de)

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**Cleaning / disinfection**

<b>Warning notice:</b>	The surfaces must not be treated with corrosive cleaning agents or disinfectants. Damage to the surface may lead to leakage, but in any case to reduced flexibility of the film. The product loses its intended properties and becomes unusable.
<b>Restrictions:</b>	The products must <b>not</b> be cleaned and disinfected thermally or by machine.
<b>First handling:</b>	Application-specific (Germany: MPBetreibV), in accordance with the relevant authorities.
<b>Cleaning preparation:</b>	Spread out on a flat surface with as little wrinkles as possible
<b>Manual cleaning:</b>	<p>It is recommended to clean / reprocess foil items immediately after they have been used, as blood and colored disinfectants are extremely aggressive. Discoloration can no longer be completely removed after a longer exposure time.</p> <p>Equipment: clear water, mild soapy water, soft brush or sponge, disposable cloths (according to requirements).</p> <ol style="list-style-type: none"> <li>1. Remove surface contamination with clear water and a soft brush, sponge or lint-free cloth. Treat all surfaces. Rinse with clean water.</li> <li>2. Apply mild soapy water to all surfaces with a soft brush, sponge or lint-free cloth, wipe and rinse with clear water.</li> </ol> <p>Pay attention to areas that are difficult to access!</p>
<b>Disinfection:</b>	<p>Non-critical products. A low level of disinfection must be carried out.</p> <p>Use a commercially available wipe disinfectant solution for sensitive plastic surfaces in accordance with the manufacturer's instructions for use. Use a maximum of 2% solution. Pay attention to areas that are difficult to access!</p> <p>Spray existing uncoated / textile belt material (restraint systems) with the disinfectant solution, moisten sufficiently. Observe the manufacturer's information on the exposure time! A book must be kept of the disinfection carried out (disinfection protocol)</p>
<b>Drying:</b>	<p><u>After cleaning:</u> wipe with dry disposable cloths, air dry / allow residual moisture to evaporate.</p> <p><u>After disinfection:</u> do not rub dry! Adhere to the exposure time for the disinfectant specified by the manufacturer! Allow residual moisture to dry off / evaporate. Wipe with a clean cloth (e.g. disposable cloth) only if necessary (emergency).</p>
<b>Maintenance:</b>	The products do not have any components that are subject to usage-related wear. Maintenance is therefore not required.
<b>Control / functional test:</b>	Regular inspections in accordance with the respective competent authority (Germany: MPBetreibV).
<b>Packaging:</b>	If necessary, storage in plastic bags of the appropriate size or in a transport bag made of nylon fabric (available as an accessory).
<b>Sterilization:</b>	Not required as it is not used in areas that are to be kept sterile.
<b>Storage:</b>	After complete drying at room temperature, avoid direct sunlight.
<b>Additional information:</b>	Not mandatory.